



# Changzhou Medical Bioengineering Co., Ltd

Add: 117, West Chenjiatou, Heping Village, Zhenglu, Tianning Dist., 213114, Changzhou, China  
Tel:86 519 88966750、88963523 Fax:86 519 88964890  
Http:// www.china-changsheng.com E-mail: changsheng@china-changsheng.com

CE 0197  
ISO 13485

## Disposable Skin Stapler

It is applied in Abdominal, Gynecological, Orthopedic, and Thoracic surgery for skin closure.



W Size




R Size


### Specification

Product Code	Staple Size	Staple Quantity	Remover Code	Skin interior tissue
CSPF-15W	7.4x4.6 mm	15 pcs	CSPF-CW	More than 5 mm Conventional Use
CSPF-25W	7.4x4.6 mm	25 pcs	CSPF-CW	
<b>CSPF-35W</b>	<b>7.4x4.6 mm</b>	<b>35 pcs</b>	<b>CSPF-CW</b>	
CSPF-15R	5.4x3.5 mm	15 pcs	CSPF-CW	Less than 5 mm Ankle, Knee, Head
CSPF-25R	5.4x3.5 mm	25 pcs	CSPF-CW	
<b>CSPF-35R</b>	<b>5.4x3.5 mm</b>	<b>35 pcs</b>	<b>CSPF-CW</b>	


### Using Instruction

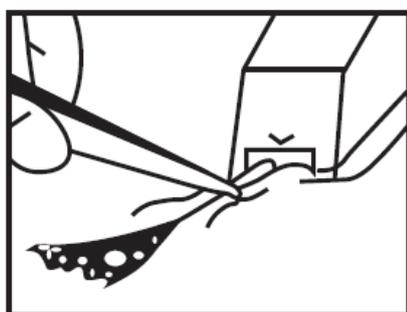

1. Evert and approximate skin edges.

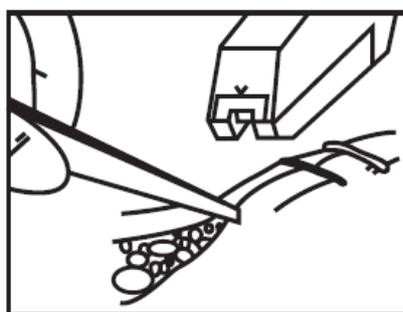

2. Position stapler over center of everted skin edges.  
Close by squeezing trigger fully until motion is halted.

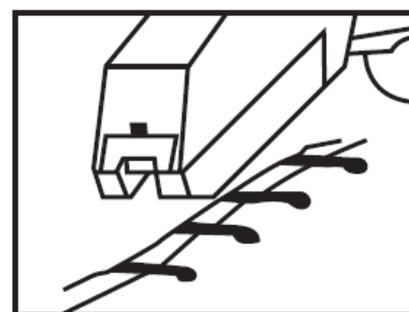

3. Release trigger fully to Achieve staple release.



## Changzhou Medical Bioengineering Co., Ltd

Add: 117, West Chenjiatou, Heping Village, Zhenglu, Tianning Dist., 213114, Changzhou, China  
Tel:86 519 88966750、88963523 Fax:86 519 88964890  
Http:// www.china-changsheng.com E-mail: changsheng@china-changsheng.com

---

### Hospital Departments

General surgical operation, Sardiathoracic surgery, Plastic surgery, Neurosurgery operation, Gynecologic surgery, Burn surgery, Emergency operation and Field rescue.

### Postoperative Effect



One week after operation  
(Prepare to remover the staples)



Three months after operation  
(Small scar after operation)

### Features:

Small stitched wound, avoid injury to epidermal tissue again and well for wound healing.

Smooth stitched wound and small scar after operation.

Shorten time for operation and decrease the pain for patients.

Reduce the workload of doctors and nurses and improve operation efficiency.

### Attention:

It is not suitable for the incision of the operation, which is required for docimasia after operation.

It is prohibited to use the disposable skin stapler in suture of interior tissues.

The product is prohibited to use when the package is broken.

The product is prohibited to use when it exceed the valid date.

The product is sterilized and discard for single use, repeat use is prohibited.

The operation doctor should be trained.



## Changzhou Medical Bioengineering Co., Ltd

Add: 117, West Chenjiatou, Heping Village, Zhenglu, Tianning Dist., 213114, Changzhou, China

Tel:86 519 88966750、88963523

Fax:86 519 88964890


Http:// www.china-changsheng.com

E-mail: changsheng@china-changsheng.com

---

# Disposable Skin Stapler Remover

It is applied in removing the staples as the accessory of skin stapler.

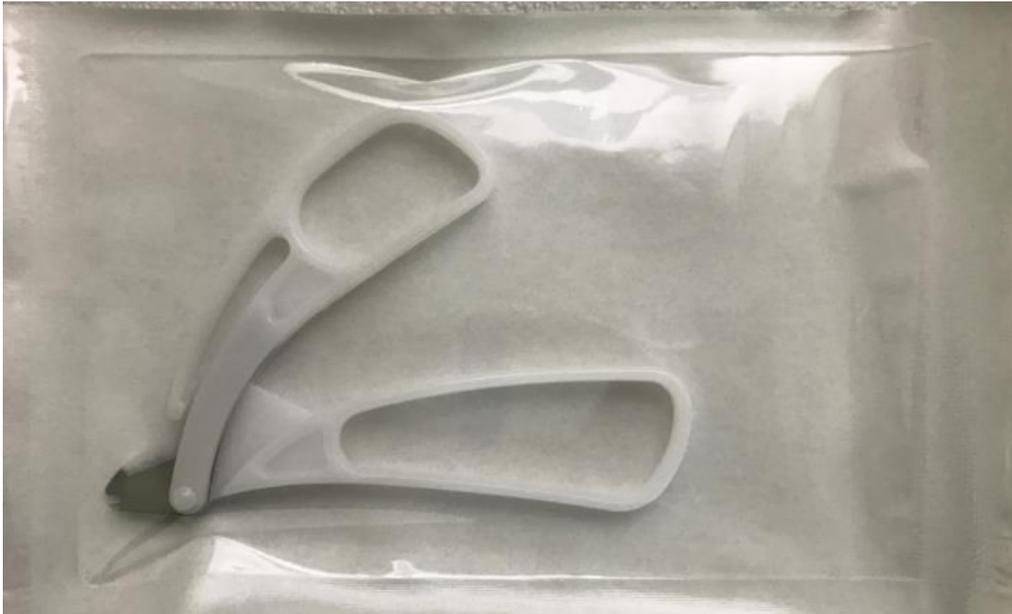

### Instruction:

To put the two parallel grapnels of the remover between the stapler pin and skin tissue and hold the remover tightly to remove the stapler pin from the skin tissue.

### Available Certificates:

CE 0197

ISO 13485

Free Sales certificate (CN & EU)

CFDA registration certificate